MobilityCharger 5051

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GB

SAFETY ASSESSMENT

Take time to read the manual. Incorrect installation of the equipment may result in damage to equipment and personal injury. If you are at all uncertain as to how to install electrical equipment: please contact us, one of our dealers or a qualified electrician. Chargers with damage on the cables shall not be used, but be returned back to dealer or producer. Do not disassemble the charger, this can cause personal injury and the warranty will no longer apply.

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CE  IP54

SAFETY ASSESSMENT

Take time to read the manual. Incorrect installation of the equipment may result in damage to equipment and personal injury. If you are at all uncertain as to how to install electrical equipment: please contact us, one of our dealers or a qualified electrician.

BATTERY CHARGER FOR WHEELCHAIRS

The charger can be placed on the floor or on a table. Connect the charging plug and mains supply plug. The charger starts/stops automatically when connected/disconnected from the wheelchair, which is why the mains plug can remain connected at all times. The charger is cleaned using a slightly moist cloth. The charger does not draw power from the grid when the wheelchair is disconnected.

The charger should be left in the original packaging until it is to be used. The charger can be transported in the wheelchair basket when using the wheelchair. It is important to protect the charger against rain when it is connected to the wheelchair and mains power supply. Recommended storage temperature is -5 to +50 ° C. Humidity 5-95% non-condensing.

Should the mains power supply cable or charger cable get damaged, do not use the charger but send it back to us or to the nearest dealer. It is essential that one does not open the charger, as it can lead to danger for both people and equipment. If the charger has been opened, the guaranty will be annulled. No changes to this product are allowed. To avoid danger of EMC noise have no cables extended.

TROUBLE-SHOOTING

Mains supply plug and charging plug connected:

No light:	Check mains connection and charging plug
Red light flashing:	Faulty battery or not fully charged. Reset the charger by removing the mains supply plug and reconnecting it. If the same indicator comes on, call a service engineer.

DO NOT COVER THE CHARGER

The charger should be left connected to the wheelchair when not in use, as well when the wheelchair is put away for an extended period of time. The charger will not start on destructively discharged batteries. Initial voltage minimum 1.45V per cell in a 24V system. The charger automatically adapts to the type of battery installed in the wheelchair.

A: CARE FOR THE ENVIRONMENT

The charger is designed to achieve as high a degree of efficiency as possible. The charger is completely recyclable.

ITEM	DESIGNATION	EXPLANATION
①	Mains indicator	Green light comes on when charger is operating
②	Battery indicator	Orange - Charging Green - Final charging. Max. charging current 2.5A Green flashing - Trickle charging Orange flashing - Conditioning repeats every 100 hours Permanent red light - Insufficient charging voltage (battery voltage below 17V or the poles of the charging cables are incorrectly connected) Red flashing - Battery voltage below 17V, flashes for 30 sec. Then permanent red light. Red/orange flashing - Conditioning has failed. The battery is faulty.
③	Mains supply plug	Plugs into 230V/50 Hz wall socket
④	Charging plug	Recommended plug for wheelchairs The charger is available in two versions; Item no.: 701239 Including Charging plug Item no.: 701240 excluding Charging plug

TECHNICAL DATA

Mains voltage [VAC]:	230	Recommended battery size [Ah]:	10-110
Mains frequency [Hz]:	50-60	Power consumption [W]:	340
Charging current [A]	10	Dimensions HxWxD [mm]:	235x175x65
Charging voltage [VDC]	28,8 -31	Ingress protection [IP]:	54
Float voltage [VDC]:	26,6	Weight [Kg]:	1,5
Altitude (kPa):	70-106	Temperature (°C):	-25 to +40
Humidity (%RH):	<95 non-condensing		

The charger is double isolated, class II.
The charger is temperature-compensated.

The charger can be used for: Valve-regulated recombination batteries, 24 V system (12 cells),
GEL/AGM batteries and open lead/acid batteries.

WARNING!

GAS - Highly explosive oxyhydrogen gas may be formed during the charging process and a small spark is enough to ignite it. For this reason you should never smoke or hold a naked flame or similar close to the battery.

ACID - The acid in the battery is corrosive and attacks clothing, metal and paint. If you spill any acid, wash it off with plenty of water. Acid sprayed into the eyes can cause serious injury: if this happens, rinse with copious amounts of water and consult a doctor.

POISON - Lead and other chemicals used in batteries are highly toxic. Wash your hands and skin thoroughly after working with batteries.

The charger must **not** be connected to non-rechargeable batteries.

ACCEPTANCE

The battery charger complies to following regulation:

93/42/EEC	Medical device directive
IEC 60601-1	Medical electrical equipment, safety
IEC 60601-1-2	Medical electrical equipment, EMC
IEC 12184	Electrically powered wheelchairs, scooters and their chargers
IEC 60529	Enclosure classification, Ingress protection
ISO 14971	Risk management
ISO 7176	Power and control systems for electric wheelchairs

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DEFA AB c/o Business
Center Hagalund AB
International GmbH